Diamondbritetm

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Diamondbritetm

ENG MICROHYBRID LIGHT CURE RESTORATIVE COMPOSITE FOR ANTERIOR AND POSTERIOR USE

INTENTION OF USE: High aesthetic restorations filling teeth cavities.

COMPOSITION: Bis-GMA, TEGDMA, Radiopaque filler, Pigments.

INSTRUCTIONS FOR USE:

CLEANING

Since cleaning and etching may cause distinct changes in the natural color of the tooth the selection of the appropriate color should be done prior cleaning and etching tooth. Perform a prophylaxis upon surface enamel to be etched, using water and pumice. You should use a prophylactic paste that contains no acid or fluorides. The soft portions of dentine and all decay should be removed. Isolate the tooth with a rubber dam or a roll of cotton. Put a protective base of zinc phosphate or calcium hydroxide.

CAVITY PREPARATION

Prepare the cavity remove the caries. If liners are required, Diamondbrite Microhybrid Composite filling material is compatible calcium hydroxide and zinc phosphate cement. Etch the enamel surface in the apply layer of bonding prior the application of the composite.

ETCHING TOOTH SURFACE ENAMEL

Apply etchant over the surface enamel of tooth using a cotton pellet or brush. Allow to remain for 40 seconds. On primary, tooth high in fluoride, a 90 – 120 seconds. If glass ionomer base is used, do not etch for more than 20 – 30 seconds. Flush area completely for 20 seconds. Do not allow patient to rinse the mouth or permit saliva to touch etched enamel surface. Air dry completely with an oil/moisture-free air source. The surface of tooth should has a dull, white, frosted appearance. If not, repeat the process allowing etchant to remain on tooth for an additional 20 – 30 seconds.

BONDING

Apply a thin uniform coating of Diamondbrite Enamel Bonding Resin immediately onto etched tooth surface. Thin the resin coating on the tooth with a gentle stream of oil/moisture-free air. Avoid contact with gingival or other soft tissue.

RESTORATION

Withdraw a suitable quantity of composite; apply the composite in layers of no more than 2 mm thick. To thicken and cure the composite expose to a visible light source, continue this procedure until desired thickness is obtained, ensure that each layer is cured separately, expose the total restoration to a further 60 seconds of light. High intensity light will require less time to cure the material so you should test to find the exposure time suitable for your type of light.

FINISHING

A smooth surface can be obtained by using strip of aluminum oxide paper and disc ranging from course to super fine. You can polish using the conventional method for acrylics.

PRECAUTION

The etching solution contains a 37% phosphoric acid solution, which can be harmful to skin and eyes, in case of contact, rinse the eyes or skin immediately in plenty of water and seek medical assistance for appropriate treatment.

WARNING:

- Do not leave at reach of children.
- In case of any change on the original function of the product, stop using it and contact the manufacturer.

For Professional Dental use only.

The Diamondbrite Microhybrid Composite is not irritable to the skin. In case of contact with the eyes, wash immediately with abundant water during at least 15 minutes and if the inconveniences persist consult a doctor. In case of contact with skin wash the area with soap and water. In case of inhalation clean the nose with water and medical assistance is required. Before the use of these products always use rubber dam. Some patients can present allergy to the Diamondbrite Microhybrid Composite or any of its components. In this case suspend it use. Do not storage in temperatures above 24 °C. Close the packages immediately after to dispense the product.

Contraindication: The material must not be used when the techniques application described are not possible.